

## **4 week test report: tube dosing pump Weinmann GmbH**

### **Infratronic Solutions**

#### **Work instruction:**

**Before every setting or even control of the disinfectant, please put on protective gloves, mask, protective gown and goggles.**

**1 step: Fill 3 Litres of Aqua Dest into the disinfection tub**

**2 step: Addition of 3 % InstruCid AF = 90 ml disinfectant**

**3 step: Insert the peristaltic metering pump, flush, 15 minutes of contact time**

**4 step: Remove the screen and let it drain, wait approx. 15 minutes**

**5 step: Rinse and flush the peristaltic metering pump with Aqua Dest**

**6 step: Optical control, maybe rinse, dry**

**7 step:       function test:               OK                       not OK**

**Place:**

**Date:**

**Signature:**